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TS100 Automatic Tissue Slide Stainer

TS100 Automatic Tissue Slide Stainer is an instrument that combines the compactness of a small in line stainer to the technology of continuous loading for histology and cytology dyeing. The stainer is now equipped by new software, more powerful and easier to use.

TS100 processing chamber is made of stainless steel for easier cleaning operations, while the external structure is painted with epoxy resin and its color has been thought to be proper to laboratory environmental.

TS100 supplied with 1 basket for 50 slides and 18 bowls and caps.

- 1 loading / parking station;
- 16 staining stations (max n.);
- 1 washing bowls with water flow.

TS100 equipped with a control unit which allows automatically standard or particular staining protocols.

TS100 staining protocols and reagents are stored in the control unit and specific software allows easy to use

In particular these following operations are managed:

- staining time of each station: 0 – 59 min
- Automatic handling of basket;

Baskets are lifted and moved from a station to another one through an electronic system.

- Adjustable dripping times in each station: 0 – 30 sec adjustable
- To prevent the reagents contamination, stops are foreseen to allow basket dripping before moving to another station
- Exhaust fan to eliminate toxic reagent vapors (option)
- Internal lamp to check reagent levels on each station (option)

Technical specifications:

- Power: 500 W
- Voltage: AC 230 V
- Frequency: 50 Hz
- Operating temperature range: from + 5 °C to + 40 °C
- Maximal humidity percentage: max 80